INSTRUCTION MANUAL EN

[Name] Particle filtering half mask

[Model] KF-A F10 (SC)

[Class] EN 149:2001 +A1:2009 FFP2 NR

[Applicable standard] EN 149:2001 +A1:2009

[Specification] 23 cm x 12cm

[Product description]

This product consists of PP nonwoven fabric, PP melt-blown fabric, spunlaced nonwoven fabric, nose clip and ear loops. This is a particle filtering half mask, which is intended to provide adequate sealing on the face of the wearer against the ambient atmosphere when the skin is dry or moist and when the head is moved. This mask is designed to protect against both solid and liquid aerosols. This mask is non-re-usable particle filtering half mask. This mask is intended exclusively for the personal protection; it is not medical face mask.

[Intended Use]

This product, which is normally used in the general working environment, is designed to provide reliable respiratory protection and against certain airborne particles and dust, block body fluids and so on.

Please read these instructions thoroughly before use.

[How to use the respirator]

1. Ensure that your face is clean-shaven or long hair is tied back, and jewelry is removed to not interfere with the face's seal.
2. Wash your hands with soap and water or use an alcohol-based hand sanitizer before touching the respirator.
3. Ensure you are using a clean respirator. Inspect the respirator for damage before each use.
4. With the reverse side up, separate top and bottom panels form a cup shape. Bend slightly at the center of the nose clip.
5. Ensure both panels are fully unfolded.
6. Hold the respirator in one hand under the chin, with the open side towards the face and nosepiece up. Hold the ear-loops and place them around each ear one by one.
7. Place your fingertips from both hands at the top of the metal nosepiece. Using two hands, mold the nose area to your nose's shape by pushing inward while moving your fingertips down both sides of the nosepiece. (Pinching the nosepiece using one hand may result in an improper fit and less effective respirator performance. Use two hands.)
8. Adjust top and bottom panels for a comfortable fit, ensuring panels are not folded in.
9. Perform a User Seal Check before each wearing. To check the respirator-to-face seal, place both hands completely over the respirator and exhale sharply. Be careful not to disturb the position of the respirator. If air leaks around the nose, re-adjust the nosepiece as described in step 6. If air leaks at the respirator edges, work the straps back along the sides of your head. If you CANNOT achieve a proper seal, DO NOT enter the contaminated area.

[Precautions]

1. This mask marked "NR", shall not be used for more than one shift.
2. Never substitute, modify, add, or omit parts in the configuration as specified by the manufacture.
3. This mask helps protect against certain particulate contaminants but does not completely eliminate exposure to the risk of contracting disease or infection.
4. Do not use the particle half mask with facial hair or any other conditions that may prevent a good face-seal, the requirements of leakage will not be achieved.
5. Discard and replace the mask if:
 - a) The mask is removed while in the contaminated areas.
 - b) Clogging of the mask causes breathing difficulties.
 - c) The mask becomes damaged.

[Shelf life] 2 years

[Storage and transportation conditions] Keep masks in the package away from direct sunlight or contaminants until use. Ambient temperature between -30°C to +70°C, and relative humidity <80%, no corrosive gas, good ventilation.

During transportation, keep away from moisture, light and heat.

[Package] 10pcs/bag

[Production date] See package

[Batch code] See package

GUANGDONG KINGFA SCI. & TECH. CO., LTD.

No.28 Delong Avenue, Shijiao Town, Qingcheng District, Qingyuan City,
Guangdong Province, China.

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Lotus NL B.V.