Atemschutzmaske FFP2 / Respirator mask FFP2

Produktdaten / Description

REF	Typ /type	Farbe / Color	Packung / Package	Karton / Carton	Palette / Pallet
HA-01D	HA PP02	weiß / white	Einzeln verpackt / packed individually 20 Stück pro Box / 20 pcs per box	20 Boxen / 20 boxes	16 Karton / 16 carton

Allgemeine Informationen / General information

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Important Note: Non-observance of the information, especially in view of use, compatibility and durability may lead to damage to persons and / or material. Meditrade does not take any liability in case of improper use. This is a single use product – changes in material / infection risk cannot be excluded in case of multiple use. All information included correspond with the last document amendment date. Errors, literal mistakes and changes expected. In case of doubt please seek expert advice. Please find any up-dated product information on our homepage www.meditrade.de

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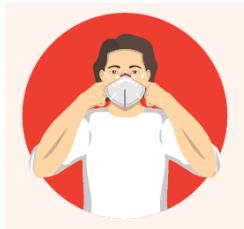
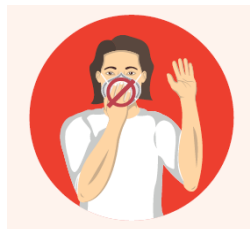
Stand / Status:
2021-04

Instructions for Use

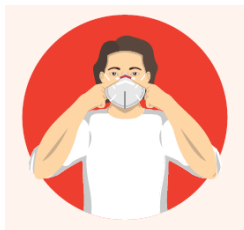
1. Wash your hands using soap and water or hand sanitizer

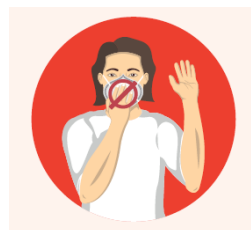
2. Pull the loops of the mask back to fit them around your ears

3. Pull the upper and lower ends of the mask over nose and chin so that no open gaps remain between face and mask

4. Firmly press the nose clip in your mask across nose and cheeks to fit

5. Avoid touching the mask while wearing

Warning

1. Please check the mask for visible damage, dirt and the expiry date before use.
2. Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of protective respirator masks and result in sickness or death.
3. A properly selected protective respirator mask is essential to protect your health. Before using this protective respirator mask, consult an industrial hygienist or occupational safety professional to determine the suitability for your intended use. Always check the protective respirator mask for fit prior to use.
4. This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this protective respirator mask when oxygen concentration is less than 19.5%.
5. Do not use when concentrations of contaminants are immediately dangerous to health or life. Do not use this protective respirator mask in explosive environment. Leave work area immediately and return to fresh air if, a) breathing becomes difficult, or b) dizziness or other distress occurs;
6. Facial hairs or beards and certain facial characteristics may reduce the effectiveness of this protective respirator mask.
7. Never alter or modify this protective respirator mask in any way.
8. Do not use for more than one shift (8 hours). No maintenance necessary.
9. Discard the used protective respirator mask after single use.
10. Keep protective respirator mask in the packaging box away from direct sunlight until use.

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